


# Set a Temp Basal Rate

## for the t:slim X2 Insulin Pump




UNITS OF MEASURE MMOL/L


The instructions below are provided as a reference tool for caregivers who are already familiar with the use of an insulin pump and with insulin therapy in general. Not all screens are shown. For more detailed information on the operation of your Tandem insulin pump, please refer to its user guide.

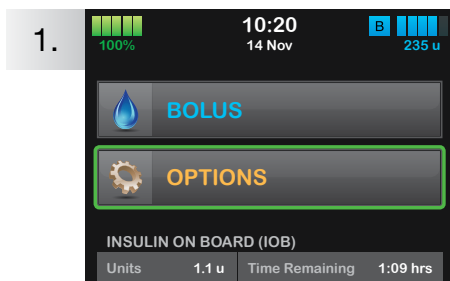


1. Tap **OPTIONS**.

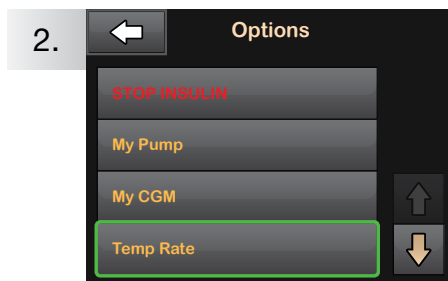

2. Tap **Temp Rate**.

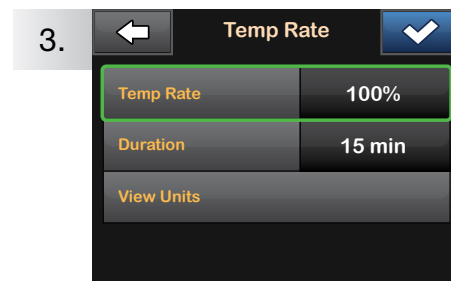


3. Tap **Temp Rate**.

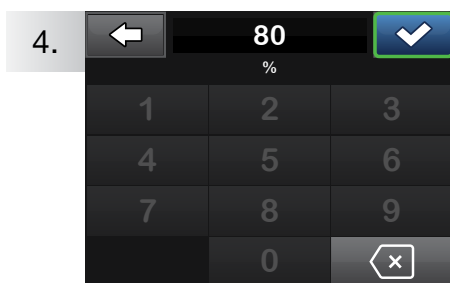


4. Using the onscreen keypad enter desired percentage. Tap .

**NOTE:** Current rate is 100%. An increase is greater than 100% and a decrease is less than 100%.

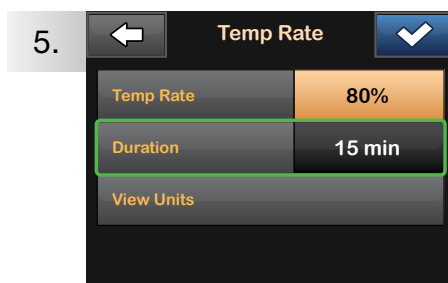

5. Tap **Duration**. Using the onscreen keypad enter desired length of time for Temp Rate. Tap .

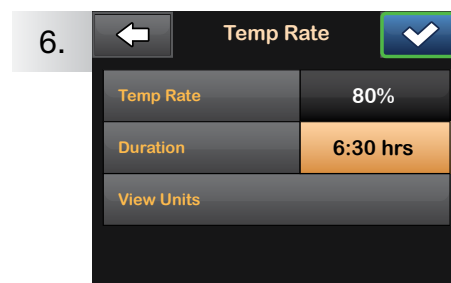


6. Verify settings and tap .

**NOTE:** To see the actual units to be delivered, tap **View Units**.




7. The **TEMP RATE STARTED** screen will appear to confirm the Temp Rate has started.

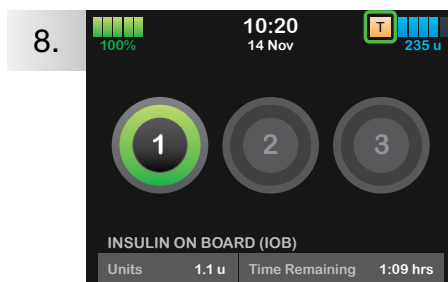

8. The Screen Lock screen will appear with the orange T icon indicating a Temp Rate is active.

**NOTE:** If a Temp Rate of 0% is currently active, the orange T icon will be replaced with a red T icon.

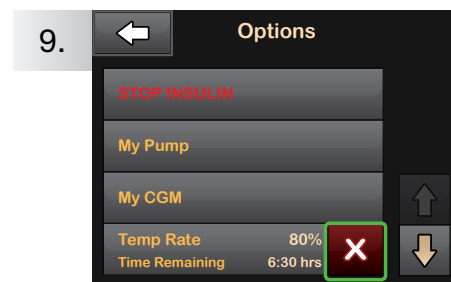

9. To stop Temp Rate at any time, tap **OPTIONS**, then tap . A confirmation screen will appear. Tap .



**TANDEM**  
DIABETES CARE

(833) 509-3598  
tandemdiabetes.ca



**t:simulator App**  
A demo as simple as our pumps!



FOR USE IN  
CANADA ONLY

