Instructions for Use

Instrucciones de uso \ Mode d'emploi /
Anwendungshinweise / Istruzioni per l'uso /
Gebruiksaanwijzing / Brugsanvisning / Käyttöohje /
Bruksanvisning / Bruksanvisning / Instrukcja użycia /
Návod k použití / Használati útmutató / Navodila
za uporabo / إرشادات الاستخدام / Indicações de utilização /
Инструкция по применению

11075 Roselle Street • San Diego, California 92121 • USA
+1-858-255-6269 • www.tandemdiabetes.com

© 2018 Tandem Diabetes Care, Inc. All rights reserved.
Tandem Diabetes Care, TruSteel, t:case and t:lock are either registered
trademarks or trademarks of Tandem Diabetes Care, Inc. in the
United States and/or other countries. AW-1000232_B

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Attention: see accompanying document

Atención: Consulte la documentación adjunta / Attention: voir
documents / Achtung: siehe Begleitdokumentation / Attenzione:
consultare la documentazione di accompagnamento / Let op: zie het
bijbehorende document / Bemærk: se medfølgende dokument / Huomio:
katso ohjeasiakirja / NB: Se vedlagte dokumenter / Obs! Se medföljande
dokument / Uwaga: patrz dokumenty dodatkowe / Pozor: Viz příložená
dokumentace / Figyelem: Olvassa el a mellékelt dokumentumot /
Pozor: glejte priloženi dokument / السيد، انظر ملف المرفق / Atenção: consulte o
documento anexo / Внимание: см. сопутствующую документацию

Rx Only Prescription required, does not apply in all geographies

Se requiere receta médica, no se aplica en todas las áreas
geográficas / Soumis à prescription (ne concerne pas tous les
marchés) / Verschreibungspflichtig, gilt nicht für alle Regionen /
Prescrizione medica richiesta, non si applica in tutte le aree
geografiche / Uitsluitend op voorschrift, niet in alle landen van
toepassing / Receptpligtig; gælder ikke i alle lande / Recepti
vaaditaan, ei päde kaikilla alueilla / Recept påkrævet, gælder ikke i
alle geografiske områder / Receptbelagt läkemedel, gäller inte i alla
områden. / Produkt dostępný na receptę; nie dotyczy niektórych
lokalizacji. / Je vyžadován lékařský předpis; nepatří ve všech zemích
/ Receptre kapható (nem minden országban) / Samo na recept,
ne velja za vsa območja / يتم توفير وصفة لا يطبق هذا في جميع المناطق المرشحة /
Requer receita médica, não se aplica a todas as áreas geográficas /
Необходимо предписание врача (относится не ко всем регионам)

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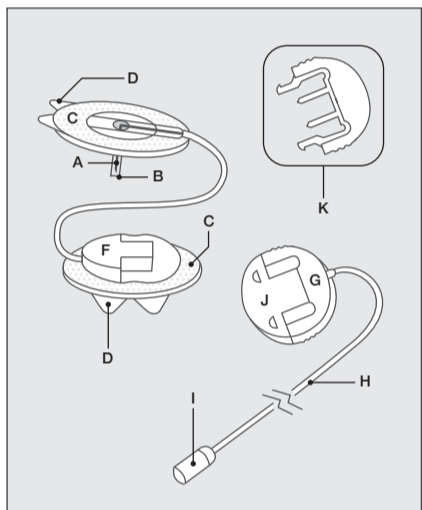
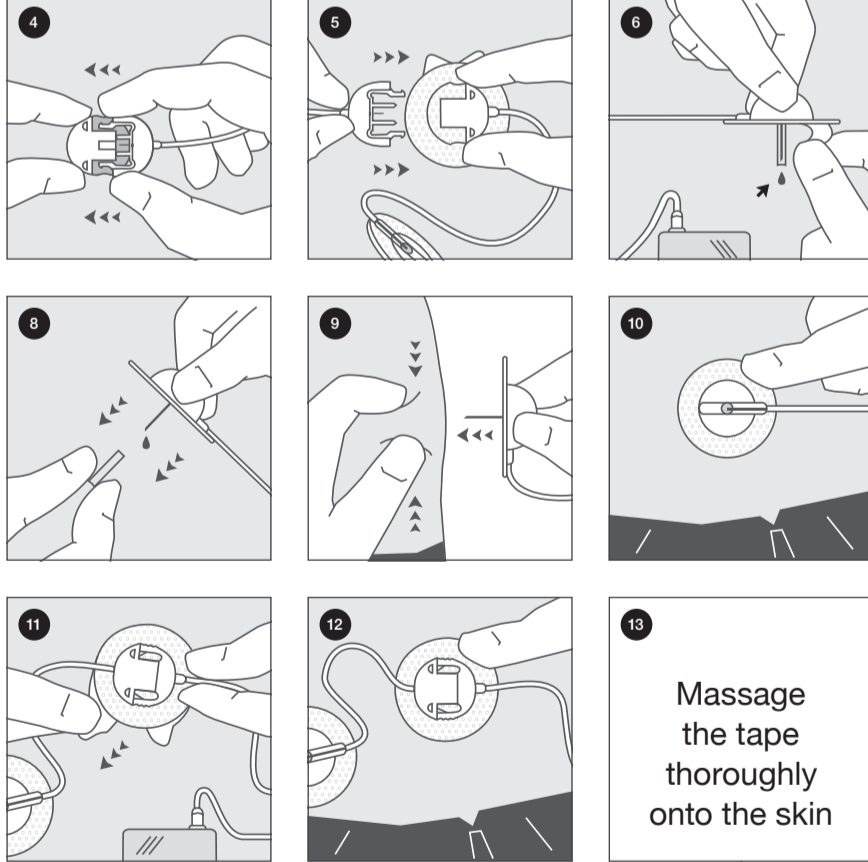
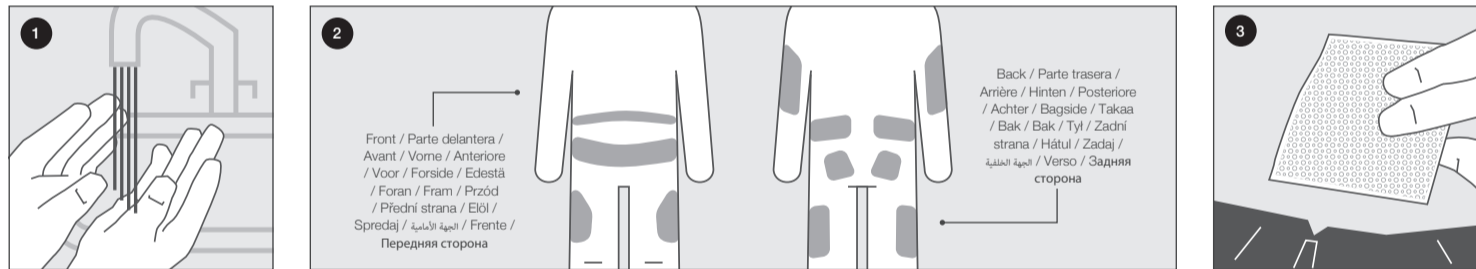
7 Prime infusion set until medication dispenses from the tip. Ensure there are no air bubbles in the tubing or TruSteel. Consult your pump manufacturer's instructions for use for proper priming procedures.

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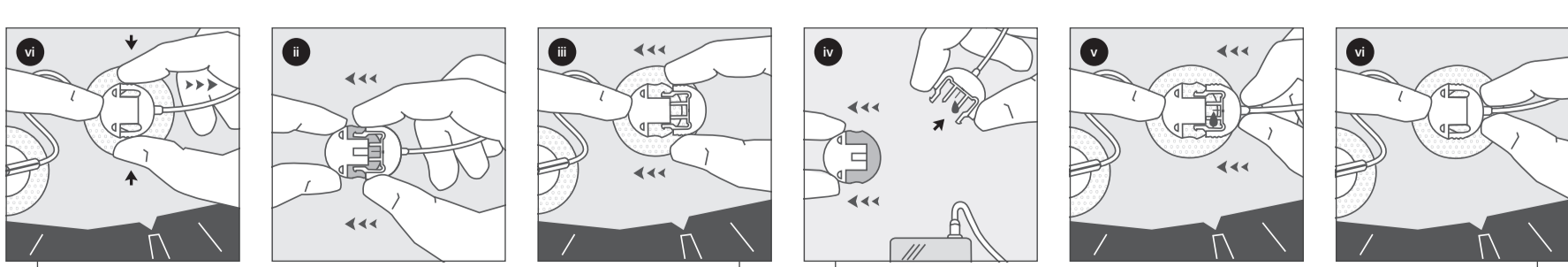
Indications for Use

The TruSteel™ infusion set is indicated for subcutaneous infusion of insulin administered by an external pump.

DESCRIPTION

TruSteel is a 90-degree, stainless steel needle infusion set with a tubing connection.

CONTRAINDICATIONS

TruSteel is neither intended nor indicated for intravenous infusion (I.V.) of medication, including blood or blood products.

WARNINGS

- TruSteel infusion sets with tlock™ tubing connectors should only be used with Tandem cartridges featuring the tlock connector.
- The TruSteel infusion set is only sterile and non-pyrogenic if the packaging is unopened and undamaged. Do not use it if the packaging is already opened or is damaged.
- Go through instructions for use carefully before inserting TruSteel, as failure to follow instructions may result in pain or injury.
- Inaccurate medication delivery, infection and/or site irritation may result from improper insertion and maintenance of infusion site.
- When using TruSteel for the first time, do so in the presence of a healthcare provider.

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- Reuse of the infusion set may cause infection, site irritation, or damage to the cannula/needle. A damaged cannula/needle may lead to inaccurate medication delivery.
- Carefully remove the needle guard before inserting the infusion set.
- Do not bend the needle prior to insertion, and do not use the infusion set if the needle is bent or has been damaged.

PRECAUTIONS

- Never prime the infusion set or attempt to free a clogged tubing while the infusion set is inserted. You may accidentally inject too much medication.
- Check the infusion set frequently to ensure that the steel needle remains firmly in place. If the steel needle is not in place, change to a new infusion set.
- TruSteel is a single-use item which must be disposed of after usage. For available sharps containers, please consult your local pharmacy. Do not clean or re-sterilize.
- Choose insertion sites as recommended by your healthcare provider.
- Do not in any way put disinfectants, perfumes, deodorants or other products containing disinfectants in contact with the tubing connector and the tubing, as these may affect the integrity of the infusion set.
- Replace with a new infusion set if the adhesive tape becomes loose.

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- Change the infusion set every 24-48 hours, or as recommended by your healthcare provider.
- Dispose of used TruSteel safely, so that no one can prick or injure themselves with it.
- When removing the infusion set, make sure that the needle is still attached to the adhesive and pulled completely out of the subcutaneous tissue (skin). If the needle is missing, immediately contact your healthcare provider for assistance.
- If infusion site becomes inflamed, replace with a new infusion set and use new site until the first site has healed.
- Use aseptic techniques when temporarily disconnecting TruSteel, and seal the coupling housing with the cover provided. Consult with healthcare provider on how to compensate for missed medication when disconnected.
- Protect the TruSteel from direct sunlight. Store at room temperature.

RECOMMENDATIONS

- If blood glucose level becomes unexplainably high, or occlusion alarm occurs, check for clogs and leaks. If in doubt, change infusion set. Discuss plan for rapid replacement of insulin with healthcare provider should any of these problems arise. Test blood glucose level to make sure problem is corrected.
- Check the blood glucose level 1-2 hours after introducing TruSteel and check the infusion site several times a day, and measure your blood glucose regularly.

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- Do not change infusion set just prior to bedtime, unless blood glucose can be checked 1-2 hours after insertion.
- Carefully monitor blood glucose levels while disconnected and after reconnecting.

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