STERILE EO Sterilized using ethylene oxide

Esterilizado con óxido de etileno / Stérilisé à l'oxyde d'éthylène / Mit Ethylenoxid sterilisiert / Sterilizzato con ossido di etilene / Gesteriliseerd met ethyleenoxide / Steriliseret ved brug af ethylenoxid / Steriloitu käyttämällä etyleenioksidia / Steriliseret med etylenoxid / Steriliserad med etylenoxid / Sterylizowany przy użyciu tlenku etylenu / Sterilizováno etylénoxidem / Αποστειρώθηκε με χρήση αιθυλενοξειδίου / Etilén-oxidál sterilizálva / Sterilizirano z etilen oksidom / 산화 에틸렌을 이용한 멸균처리 / 已使用环氧乙烷灭菌 / تم التطهير باستخدام أوكسيد الإيثيلين / Esterilizado com óxido de etileno / Стерилизовано при помощи оксида этилена

GTIN Global Trade Item Number

Número de artículo de comercio mundial / Code article international (GTIN) / GS1-artikelnnummer / Codice GTIN (Global Trade Item Number) / Global Trade Item Number / Globalt handelsvarenummer / Kansainvälinen kauppatavarannumero / Globalt handelsartikelnnummer / Global Trade Item Number / Globalny Numer Jednostki Handlowej / Globální číslo obchodní položky / Διεθνής κωδικός εμπορικών μονάδων / Globális cikkszám / Globalna trgovinska številka izdelka / 국제거래단품식별코드 / 全球貿易項目編號 / رقم الصف التجاري العالمي / Número de item de comércio global / Глобальный номер единицы товара

Use by (year - month - day)

Utilizar antes de (año - mes - día) / Date de péremption (année - mois - jour) / Verfallsdatum (Jahr - Monat - Tag) / Utilizzare entro (anno - mese - giorno) / Te gebruiken vóór (jaar - maand - dag) / Sidste anvendelsesdag (år - måned - dag) / Käytettävä viimeistään (vuosi - kuukausi - päivä) / Brukes innen (år - måned - dag) / Utgångsdatum (år - månad - dag) / Zużyc przed (rok - miesiąc - dzień) / Datum spotřeba (rok - měsíc - den) / Χρήση έως (έτος - μήνας - ημέρα) / Felhasználható (év - hónap - nap) / Rok uporabe (leto - mesec - dan) / 사용기한 (연 - 월 - 일) / 有效期 (年 - 月 - 日) / الاستعمال (يوم - شهر - سنة) / Utilizar até (ano - mês - dia) / Годен до: (год-месяц-день)

AutoSoft™ 90

EN	English	4
ES	Español	8
FR	Français	12
DE	Deutsch	16
IT	Italiano	20
NL	Nederlands	24
DA	Dansk	28
FI	Suomi	32
NO	Norsk	36
SV	Svenska	40
PL	Polski	44
CS	Česky	48
EL	Greek	52
HU	Magyar	56
SL	Slovenščina	60
KO	한국어	64
ZH	中文	68
AR	عربي	72
PT	Português	76
RU	Русский	80

Indications for Use

The AutoSoft™ 90 infusion set is indicated for the subcutaneous infusion of insulin administered by an external pump.

DESCRIPTION

AutoSoft 90 is an infusion device which combines an infusion set (catheter) with an aid for insertion (insertor). As the infusion set is an integrated part of the inserter, AutoSoft 90 is delivered assembled and ready for use as a single, sterile unit.

CONTRAINDICATIONS

AutoSoft 90 is neither intended nor indicated for use in intravenous infusion of blood or blood products.

WARNINGS

- AutoSoft 90 infusion sets with t:lock™ tubing connectors should only be used with Tandem cartridges featuring the t:lock connector.
- AutoSoft 90 is only sterile and non-pyrogenic if the insertion device is unopened and undamaged. Do not use if the insertion device is already opened or has been damaged. Ensure sterility by

checking if the sterile paper and tamper-proof seal are unbroken.

- Carefully examine the instructions for use before inserting AutoSoft 90. Failure to follow instructions may result in pain or injury.
- When using AutoSoft 90 for the first time, do so in the presence of a healthcare provider. Since different individuals have varying amounts of subcutaneous layer, ensure that an appropriate cannula length is chosen to fit your needs.
- Do not in any way put disinfectants, perfumes, deodorants, cosmetics or other products containing disinfectants in contact with the connector or the tubing. These may affect the integrity of the infusion set.
- AutoSoft 90 is a single-use device and should be disposed of immediately after use. Do not clean or re-sterilize.
- AutoSoft 90 can be disposed of safely after insertion if the lid has been placed back on. Please consult your local pharmacy for sharps containers.
- Be sure that the needle guard is removed before insertion is made.
- Release the tubing with caution as a hard pull of the tubing can result in damage to the infusion set/introducer needle. Ensure

that the infusion set is properly in place when the tubing is fully released (see figure 8 on page 84).

- Do not leave air in the infusion set. Make sure to fill AutoSoft 90 completely.
- Inaccurate medication delivery, infection and/or site irritation may result from improper insertion or maintenance of your infusion site.
- Replace the infusion set every 48-72 hours, or per your healthcare provider's instructions.
- If the soft cannula bends during insertion, discard and apply a new AutoSoft 90 immediately.
- Replace the infusion set if the adhesive tape becomes loose or is displaced from its original site. Since the cannula is soft, it will not cause any pain if it slips out, and this may take place without notice. The soft cannula must always be completely inserted to receive the full amount of medication.
- If your infusion site becomes inflamed, replace infusion set and use new site until the first site has healed.
- Do not re-insert the introducer needle into the infusion set. This could cause a tearing of the soft cannula and unpredictable medication flow.

- Never try to fill or free clogged tubing while the infusion set is inserted. This may result in unpredictable medication flow.
- Always clean the insertion site when temporarily disconnecting the infusion set. Consult your healthcare provider on how to compensate for missed medication when disconnected. Carefully monitor blood glucose levels when disconnected and after reconnecting.
- Protect the AutoSoft 90 infusion set from direct sunlight and atmospheric humidity. Store in a dry place at room temperature.
- Never point a loaded insertion device towards any body part, where insertion is not desired.
- Reuse of the infusion set may cause infection, site irritation, or damage to the cannula/needle. A damaged cannula/needle may lead to inaccurate medication delivery.
- When using insulin, do not change your infusion set just prior to bedtime, unless blood glucose can be checked 1-2 hours after insertion.
- When you are infusing insulin and your blood glucose level becomes unexplainably high or an occlusion alarm occurs, check for clogs and leaks. If in doubt, change your infusion set since the soft cannula could be dislodged, crimped or partially clogged. Discuss a plan for rapid replacement of insulin with your healthcare provider should any of these problems arise. Test your blood glucose level to make sure the problem is corrected.
- Carefully monitor your blood glucose level when disconnected and after reconnecting.

RECOMMENDATIONS

- Check your blood glucose level 1-2 hours after introducing AutoSoft 90. Measure your blood glucose on a regular basis. Discuss this with your healthcare provider.

Preparation and Insertion

Read all instructions carefully before using the AutoSoft 90 infusion set. For information about pump therapy, possible errors, and potential risks and their causes, consult your pump manufacturer's instructions for use before connecting AutoSoft 90. When using AutoSoft 90, follow proper hygiene procedures. Please see the corresponding illustrations on page 84-86.

CONTENTS

- A. Lid
- B. Needle guard
- C. Soft cannula
- D. Adhesive
- E. Cannula housing
- F. Connector needle
- G. Tubing
- H. Introducer needle
- I. Insertion device
- J. Sterile paper
- K. Three raised dots – press here when lifting the lid off (see figure 6)
- L. Lined indentations – press here when loading AutoSoft 90 (see figure 10)
- M. Round indentations – press here when inserting AutoSoft 90 (see figure 12)
- N. Disconnect cover (see figure ii)

INSTRUCTIONS

1. Wash your hands before inserting AutoSoft 90.
2. Recommended insertion areas. Do not use an area right next to your previous place of insertion.
3. Clean the area of insertion with a disinfectant as directed by your healthcare provider. Be sure the area is dry before inserting AutoSoft 90.
4. Pull the red tape to remove the seal.
5. Remove the sterile paper.
6. Press the three raised dots on each side of the lid firmly with one hand and lift up with the other hand.
CAUTION: Be careful not to bend or touch the AutoSoft 90 introducer needle when preparing the inserter and infusion set.
IMPORTANT NOTE: If the infusion set is not securely placed in the insertion device with the needle pointing straight ahead prior to insertion, pain or minor injury may occur. To avoid this, please ensure that the following steps are taken gently so the infusion set is not loosened / removed by accident.
7. **Unwinding the tubing:** Carefully twist the beginning of the tubing out of the slot. Pull up gently. Now unwind the tubing from the insertion device by pulling it carefully upwards.

CAUTION: Do not touch the AutoSoft 90 introducer needle when unwinding the tubing. Be careful not to pull too hard when the tubing is fully released. You could accidentally remove the infusion set from the introducer needle.

8. Connect AutoSoft 90 to a reservoir/ cartridge and the pump. Prime infusion set. Ensure there are no air bubbles in the tubing or AutoSoft 90.

CAUTION: When priming the infusion set, hold AutoSoft 90 with the needle pointing down to ensure that medication does not come in contact with the adhesive backing paper.

9. Pull up gently to remove the adhesive backing paper.
10. **Prepare the inserter:** Place fingers on lined indentations. Press lined indentations on each side. Pull spring up until you hear a “click”.
11. Carefully remove the needle guard by gently twisting and then pulling it.
CAUTION: Before positioning AutoSoft 90 over clean insertion site, place the tubing in the slot to ensure that the tubing is not caught underneath the device during insertion.

- 12. Insert Autosoft 90:** Position AutoSoft 90 over the insertion site. Simultaneously press the round indentations on both sides of the insertion device to insert AutoSoft 90.
13. Push gently on the center of the inserter to secure the adhesive on the skin.
14. Remove the inserter/introducer needle by grasping the center and pulling it gently back. Massage the tape thoroughly onto the skin. Fill the empty space in the cannula with the following amount of insulin:

U-100 INSULIN:

6 mm: 0.3 units (0.003 ml)

9 mm: 0.5 units (0.005 ml)

15. Push the lid back in place until you hear a “click.”

DISCONNECT

- i. AutoSoft 90 allows you to temporarily disconnect your pump without having to change the infusion set. Hold the adhesive in place by placing a finger just in front of the cannula housing, and gently squeeze the sides of AutoSoft 90. Pull the connector needle straight out from the cannula housing.

- ii. Insert the disconnect cover into the cannula housing until you hear a “click”.

RECONNECT

- iii. Prime the infusion set until medication dispenses from the needle.
CAUTION: When priming the infusion set, hold AutoSoft 90 with the needle pointing down.
- iv. Place a finger on the adhesive tape just in front of the cannula housing.
- v. Push the connector needle straight in until you hear a “click”.

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